Hb-Hp-Komplex ELISA Kit

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4. Chalkias, A. et al., 2011. Patients with colorectal cancer are characterized by increased concentration of fecal hb-hp complex, myeloperoxidase, and secretory IgA. *American journal of clinical oncology*, **34**(6), pp.561–6.

Verwendete Symbole:

Temperaturbegrenzung

Bestellnummer

In-Vitro-Diagnostikum

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Hersteller

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Hb/Hp complex ELISA Kit

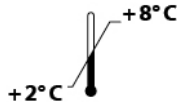
For the in vitro determination of hemoglobin/haptoglobin complex in stool

EU: IVD / CE

US: Research Use Only. Not for use in diagnostic procedures.

Valid from 19.05.2014

REF K 7837D

IVD **CE**

Immundiagnostik AG, Stubenwald-Allee 8a, D-64625 Bensheim

Tel.: +49 6251 70190-0

Fax: +49 6251 849430

e.mail: info@immundiagnostik.com

www.immundiagnostik.com

Table of Contents

1. INTENDED USE	14
2. INTRODUCTION	14
3. MATERIAL SUPPLIED	14
4. MATERIAL REQUIRED BUT NOT SUPPLIED	15
5. PREPARATION AND STORAGE OF REAGENTS	15
6. STORAGE AND PREPARATION OF SAMPLES	16
<i>Extraction of the stool samples</i>	16
<i>Sample stability and storage</i>	17
7. ASSAY PROCEDURE	17
<i>Principle of the test</i>	17
<i>Test procedure</i>	17
8. RESULTS	18
9. LIMITATIONS	19
10. QUALITY CONTROL	19
<i>Reference range</i>	20
11. PERFORMANCE CHARACTERISTICS	20
<i>Precision and reproducibility</i>	20
<i>Analytical Sensitivity</i>	20
12. PRECAUTIONS	20
13. TECHNICAL HINTS	21
14. GENERAL NOTES ON THE TEST AND TEST PROCEDURE	21
15. REFERENCES	22
<i>General literature</i>	22
<i>Literature using Immundiagnostik Hb/Hp complex ELISA</i>	22

1. INTENDED USE

The Immundiagnostik assay is intended for the quantitative determination of hemoglobin/haptoglobin complex in stool. For *in vitro* diagnostic use only.

2. INTRODUCTION

In contrast to commercially available rapid tests, this hemoglobin/haptoglobin complex ELISA does not require previous adherence to a diet (no raw meat etc.) and recognizes human hemoglobin/haptoglobin complex in 100-fold lower concentrations. This avoids false-negative results. Because of the antibodies used, false-positive results are almost excluded. Recent data show that the hemoglobin/haptoglobin complex determination increases the clinical specificity and sensitivity.

Indications

- Occult blood in stool
- Crohn's disease; Ulcerative Colitis
- Suspicion of colon carcinoma
- Polyps in the colorectum

3. MATERIAL SUPPLIED

Cat. No.	Label	Kit components	Quantity
K 7837D MTP	PLATE	Holder with pre-coated strips	12 x 8
K 7837D WP	WASHBUF	ELISA wash buffer concentrate, 10x	2 x 100 ml
K 7837D EP	IDK Extract®	Extraction buffer concentrate <i>IDK Extract®</i> , 2.5 x	1 x 100 ml
K 7837D PV	SAMPLEBUF	Dilution buffer, ready to use	2 x 15 ml
K 7837D K	CONJ	Conjugate, (mouse anti-human Hb, peroxidase-labeled), ready to use	1 x 15 ml
K 7837D CAL	CAL	Calibrator, lyophil.	4 vials
K 7837D HC	HC	High Control, lyophil.	4 vials
K 7837D LC	LC	Low Control, lyophil.	4 vials
K 7837D TMB	SUB	TMB substrate (tetramethylbenzidine), ready to use	1 x 15 ml
K 7837D AC	STOP	ELISA stop solution, ready to use	1 x 15 ml

4. MATERIAL REQUIRED BUT NOT SUPPLIED

- Ultra pure water*
- Laboratory balance
- Calibrated precision pipettors and 10–1000 µl tips
- Foil to cover the microtiter plate
- Horizontal microtiter plate shaker
- Multi-channel pipets or repeater pipets
- Centrifuge, 3000 g
- Vortex
- Standard laboratory glass or plastic vials, cups, etc.
- Microtiter plate reader (required filters see chapter 7)

* Immundiagnostik AG recommends the use of Ultra Pure Water (Water Type 1; ISO 3696), which is free of undissolved and colloidal ions and organic molecules (free of particles > 0.2 µm) with an electrical conductivity of 0.055 µS/cm at 25 °C (≥ 18.2 MΩ cm).

5. PREPARATION AND STORAGE OF REAGENTS

- To run the assay more than once, ensure that reagents are stored at the conditions stated on the label. **Prepare only the appropriate amount necessary for each run.** The kit can be used up to 4 times within the expiry date stated on the label.
- Reagents with a volume less than **100 µl** should be centrifuged before use to avoid loss of volume.
- The **ELISA wash buffer concentrate** (WASHBUF) should be diluted **1:10 in ultra pure water** before use (100 ml concentrate + 900 ml ultra pure water), mix well. Crystals could occur due to high salt concentration in the stock solutions. The crystals must be redissolved at room temperature or 37 °C before dilution of the buffer solutions. The **buffer concentrate** is stable at **2–8 °C** until the expiry date stated on the label. **Diluted buffer solution** (wash buffer) can be stored in a closed flask at **2–8 °C for one month**.
- The **extraction buffer concentrate** (*IDK Extract*®) must be diluted with ultra pure water **1:2.5** before use (100 ml *IDK Extract*® + 150 ml ultra pure water), mix well. Crystals could occur due to high salt concentration in the stock solutions. Before dilution, the crystals must be redissolved at 37 °C in a water bath. The ***IDK Extract*® extraction buffer concentrate** is stable at **2–8 °C** until the expiry date stated on the label. Diluted **buffer solution** (extraction buffer) can be stored in a closed flask at **2–8 °C for four months**.
- The lyophilized **HC** (high control), **LC** (low control) and **CAL** (calibrator), must be reconstituted with **500 µl of ultra pure water**. Allow the vial content to

dissolve for 10 minutes and mix thoroughly by gentle inversion to insure complete reconstitution. **Reconstituted standards and control are not stable.**

Please dispense 50 µl of SAMPLEBUF (sample dilution buffer) as BLANK into the respective well.

- All other test reagents are ready to use. Test reagents are stable until the expiry date (see label of test package) when stored at **2–8 °C**.

6. STORAGE AND PREPARATION OF SAMPLES

Extraction of the stool samples

We recommend the following sample preparation:

Stool Sample Application System (SAS) (Cat. No.: K 6998SAS)

Stool sample tube – Instructions for use

Please note that the dilution factor of the final stool suspension depends on the amount of stool sample used and the volume of the buffer.

SAS with 1.5 ml extraction buffer:

Applied amount of stool:	15 mg
Buffer Volume:	1.5 ml
Dilution Factor:	1:100

Please follow the instructions for the preparation of stool samples using the SAS as follows:

- a) The raw stool sample has to be thawed. For particularly heterogeneous samples we recommend a mechanical homogenisation using an applicator, inoculation loop or similar device.
- b) **Fill the empty sample tube with 1.5 ml** of ready-to-use *IDK Extract*[®] extraction buffer before using it with the sample. Important: Allow the extraction buffer to reach room temperature.
- c) Unscrew the tube (yellow part of cap) to open. Insert yellow dipstick into sample. The lower part of the dipstick has notches which need to be covered completely with stool after inserting it into the sample. Place dipstick back into the tube. When putting the stick back into the tube, excess material will be stripped off, leaving 15 mg of sample to be diluted. Screw tightly to close the tube.
- d) Shake the tube well until no stool sample remains in the notches. Important:

Please make sure that you have a maximally homogenous suspension after shaking. Especially with more solid samples, soaking the sample in the tube with buffer for ~ 10 minutes improves the result.

- e) Allow sample to stand for ~10 minutes until sediment has settled. Floating material like shells of grains can be neglected.
- f) Carefully unscrew the complete cap of the tube including the turquoise ring plus the dipstick. Discard cap and dipstick. Make sure that the sediment will not be dispersed again.

The sample suspension is now ready for use.

The sample can also be used in a pipetting automat. Place the sample in the sample rack according to instrument instructions.

Sample stability and storage

Raw stool can be stored for **1 month** at **-20 °C**.

Stool extract is stable at room temperature (15-30 °C) for one day, at 2–8 °C as well as at -20 °C for seven days. Avoid more than three freeze-thaw cycles.

7. ASSAY PROCEDURE

Principle of the test

This ELISA is used for quantitative determination of hemoglobin/haptoglobin complex in stool. The hemoglobin/haptoglobin complex in the sample is bound to anti-haptoglobin antibodies (in excess), which are immobilized on the surface of the microtiter wells. To remove all unbound substances, a washing step is carried out. In a second incubation step an anti-hemoglobin peroxidase-labeled antibody is added. After another washing step, to remove all unbound substances, the solid phase is incubated with the substrate, tetramethylbenzidine (TMB). An acidic solution is then added to stop the reaction. The color converts from blue to yellow. The intensity of the yellow color is directly proportional to the concentration of the complex in the sample. Hemoglobin/haptoglobin complex, present in the patient samples, is quantified by referring the optical density of patient's samples to a lot-dependant master calibration curve. This is done by using a reference/calibrator that is run with each test.

Test procedure

Bring all **reagents and samples to room temperature** (15–30 °C) and mix well.

Take as many **microtiter strips** as needed from kit. Store unused strips covered at

2–8° C. Strips are stable until expiry date stated on the label.

We recommend to carry out the tests in duplicate.

1.	Wash the pre-coated microtiter plate 5 x with 250 µl ELISA wash buffer before use . After the final washing step, the inverted microtiter plate should be tapped on absorbent paper.
2.	Add in each well 50 µl of SAMPLEBUF (sample dilution buffer)
3.	Then 50 µl of HC/LC/CAL/BLANK/SAMPLE (high control/low control/calibrator/blank/sample) into respective well (see template given in the test kit).
4.	Incubate for 1 hour at room temperature (15–30°C).
5.	Decant the content of the plate and wash the wells 5 x with 250 µl diluted WASHBUF.
6.	Add 100 µl of CONJ (peroxidase-labeled antibody).
7.	Incubate for 1 hour at room temperature (15–30°C).
8.	Decant the content of the plate and wash the wells 5 x with 250 µl diluted WASHBUF.
9.	Add 100 µl SUB (TMB substrate).
10.	Incubate for 10–20 minutes at room temperature (15–30°C)*
11.	Add 100 µl STOP (stop solution) and mix shortly.
12.	Determine absorption immediately with an ELISA reader at 450 nm against 620 nm (or 690 nm) as a reference. If no reference wavelength is available, read only at 450 nm. If the extinction of the highest standard exceeds the range of the photometer, absorption must be measured immediately at 405 nm against 620 nm as a reference.

* The intensity of the color change is temperature sensitive. We recommend observing the color change and stopping the reaction upon good differentiation.

8. RESULTS

For result evaluation, please use a four parametric logit-log model based on the standard curve of the respective kit lot and the Calibrator value (CAL). All essential information on the standard curve is provided on the QC data sheet of the respective product lot.

The calibration curve can be expressed either by the concentration of each standard with its corresponding optical density or by the four parameters A, B, C and D. In both cases the optical density of the calibrator (CAL) is essential.

Depending on your evaluation software program, either the one or the other kind of data described above should be entered.

Caution: Please make sure that all parameters and values are transferred accurately into your software as minor deviations can cause severe errors during evaluation.

The plausibility of the duplicate values should be examined before the automatic evaluation of the results. If this option is not available with the programme used, the duplicate values should be evaluated manually.

Stool

Since the sample dilution is already considered in the calibration curve, the dilution factor is 1.

9. LIMITATIONS

Samples with concentrations above the measurement range must be further diluted and re-assayed. Please consider this greater dilution when calculating the results.

Samples with concentrations lower than the measurement range cannot be clearly quantified.

The upper limit of the measurement range can be calculated as:

highest concentration of the standard curve × sample dilution factor to be used

The lower limit of the measurement range can be calculated as:

Analytical sensitivity × sample dilution factor to be used

10. QUALITY CONTROL

Immundiagnostik recommends the use of external controls for internal quality control, if possible.

Control samples should be analysed with each run. Results, generated from the analysis of control samples, should be evaluated for acceptability using appropriate statistical methods. The results for the patient samples may not be valid if within the same assay one or more values of the quality control sample are outside the acceptable limits.

Reference range

Hemoglobin/haptoglobin complex (stool)*: **< 2 µg/ml \triangleq < 2 µg/g**
(1 g stool \triangleq 1 ml)

* H.G. Bischoff et al., 1990. Fecal Occult Blood: A Sensitive and Specific Method for Detection by Immunological Determination of Human Albumin and Hemoglobin. *Arztl. Lab.*, **36**, pp. 101–112.

We recommend each laboratory to establish its own reference range.

11. PERFORMANCE CHARACTERISTICS

Precision and reproducibility

Intra-Assay (n = 42)

Sample	Hemoglobin/ haptoglobin [µg/g]	CV [%]
1	3.88	2.49
2	8.31	3.89

Inter-Assay (n = 63)

Sample	Hemoglobin/ haptoglobin [µg/g]	CV [%]
1	1.16	4.29
2	4.2	2.64

Analytical Sensitivity

The sensitivity was set as $B_0 + 2 \text{ SD}$. The zero-standard was measured 20 times. A detection limit of 0.054 µg/ml was obtained.

12. PRECAUTIONS

- All reagents in the kit package are for *in vitro* diagnostic use only.
- Human materials used in kit components were tested and found to be negative for HIV, Hepatitis B and Hepatitis C. However, for safety reasons, all kit components should be treated as potentially infectious.
- Kit reagents contain sodium azide or Proclin as bactericides. Sodium azide and Proclin are toxic. Substrates for the enzymatic color reactions are toxic and carcinogenic. Avoid contact with skin or mucous membranes.

- The stop solution consists of diluted sulphuric acid, a strong acid. Although diluted, it still must be handled with care. It can cause burns and should be handled with gloves, eye protection, and appropriate protective clothing. Any spill should be wiped up immediately with copious quantities of water. Do not breath vapour and avoid inhalation.

13. TECHNICAL HINTS

- Do not interchange different lot numbers of any kit component within the same assay. Furthermore we recommend not assembling wells of different microtiter plates for analysis, even if they are of the same batch.
- Control samples should be analyzed with each run.
- Reagents should not be used beyond the expiration date stated on kit label.
- Substrate solution should remain colourless until use.
- To ensure accurate results, proper adhesion of plate sealers during incubation steps is necessary.
- Avoid foaming when mixing reagents.
- Do not mix plugs and caps from different reagents.
- The assay should always be performed according the enclosed manual.

14. GENERAL NOTES ON THE TEST AND TEST PROCEDURE

- This assay was produced and distributed according to the IVD guidelines of 98/79/EC.
- The guidelines for medical laboratories should be followed.
- Incubation time, incubation temperature and pipetting volumes of the components are defined by the producer. Any variation of the test procedure, which is not coordinated with the producer, may influence the results of the test. Immundiagnostik AG can therefore not be held responsible for any damage resulting from incorrect use.
- Warranty claims and complaints regarding deficiencies must be logged within 14 days after receipt of the product. The product should be send to Immundiagnostik AG along with a written complaint.

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Literature using Immundiagnostik Hb/Hp complex ELISA

4. Chalkias, A. et al., 2011. Patients with colorectal cancer are characterized by increased concentration of fecal hb-hp complex, myeloperoxidase, and secretory IgA. *American journal of clinical oncology*, **34**(6), pp.561–6.

Used symbols:

Temperature limitation

Catalogue Number

In Vitro Diagnostic Medical Device

Contains sufficient for <n> tests

Manufacturer

Use by

Lot number