CERTEST

Rotavirus- Adenovirus

ONE STEP *Rotavirus* and
Adenovirus CARD TEST

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ENGLISH

INTENDED USE

CerTest *Rotavirus-Adenovirus* one step card test is a coloured chromatographic immunoassay for the simultaneous qualitative detection of *Rotavirus* and *Adenovirus* in stool samples.

CerTest *Rotavirus-Adenovirus* card test offers a simple and a highly sensitive screening assay to make a presumptive diagnosis of *Rotavirus* and/or *Adenovirus* infection.

INTRODUCTION

Rotavirus, *Adenovirus* and *Astrovirus* are most common and major causes of severe gastroenteritis in infants and young children. Pattern also observed in adults. They are transmitted by faecal-oral contact. The main symptoms of viral gastroenteritis are watery diarrhoea and vomiting. The affected person may also have headache, fever and abdominal cramps ("stomach ache"). In general, the symptoms begin 1 to 2 days following infection with a virus that causes gastroenteritis and may last for 1 to 10 days, depending on which virus causes the illness (*Rotavirus* 3 days, *Adenovirus* 5-8 days and *Astrovirus* 3 days).

TEST PRINCIPLE

CerTest *Rotavirus-Adenovirus* is based on the principle of a qualitative immunochromatographic assay for the determination of *Rotavirus* and *Adenovirus* in stool samples.

The strip consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test lines (T), in the results window, against *Rotavirus* and *Adenovirus* and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with three label solution: mouse monoclonal antibodies anti-*Rotavirus* test label solution conjugated to red polystyrene latex, mouse monoclonal antibodies anti-*Adenovirus* test label solution conjugated to blue polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming three coloured conjugate complexes.

If the sample is *Rotavirus* positive, the antigens of the diluted sample react with the red-coloured conjugate complex (anti-*Rotavirus* monoclonal antibodies-red polystyrene microspheres) and if the sample is *Adenovirus* positive, the antigens of the diluted sample react with the blue-coloured conjugate complex (anti-*Adenovirus* monoclonal antibodies-blue polystyrene microspheres), which were previously pre-dried on the absorbent pad. The mixture then moves upward on the membrane by capillary action. As the sample flows through the test membrane, the binding conjugate complexes migrate. The anti-*Rotavirus* and anti-*Adenovirus* antibodies present on the membrane (test lines) capture the coloured conjugates and the red and the blue lines will be visible. These bands are used to interpret the result.

If the sample is negative, there is no *Rotavirus* and *Adenovirus* antigens presence and yet, the antigens may be present in a concentration lower than the detection limit value, for which the reaction will not take place with any coloured conjugate complex. The anti-*Rotavirus* and anti-*Adenovirus* antibodies present on the membrane (test lines) will not capture the antigen-red/blue-coloured conjugate complex (not formed), for which the red and the blue line will not appear.

Whether the sample is positive or not the mixture continues to move across the membrane to the immobilized specific antibodies placed in the control line. The anti-specific protein antibodies present on the membrane will capture control green-conjugate complex and the control line will always appears. The presence of this green line serves as: 1) verification that sufficient volume is added, 2) that proper flow is obtained and 3) an internal control for the reagents.

STORAGE AND STABILITY

Store as packaged in the sealed pouch at 2-30°C. The test is stable until the expiration date marked on its sealed pouch. The test must remain in the sealed pouch until use. Do not freeze.

PRECAUTIONS

- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- All the specimens should be considered potentially hazardous and handled in the same manner as an infectious agent. A new test must be used for each sample to avoid contamination errors.
- The tests should be discarded in a proper biohazard container after testing.
- Reagents contain preservatives. Avoid any contact with skin or mucous membrane. Consult safety data sheet, available on request.
- Components provide in the kit are approved for use in the CerTest *Rotavirus-Adenovirus* test. Do not use any other commercial kit component.
- Follow Good Laboratory Practices, wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area.

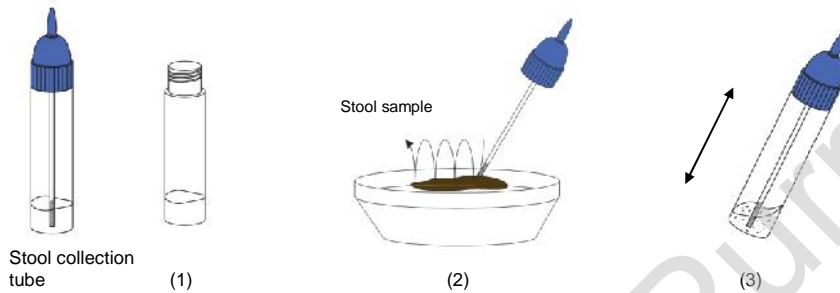
SPECIMEN COLLECTION AND PREPARATION

Stool samples should be collected in clean containers. The samples can be stored in the refrigerator (2-8°C) for 1-2 days prior to testing. For longer storage, maximum 1 year, the specimen must be kept frozen at -20°C. In this case, the sample will be totally thawed and brought to room temperature before testing. Homogenise stool sample as thoroughly as possible prior to preparation.

Specimen preparation (see illustration):

1. Take out the cap of the stool collection tube (1) and use the stick to pick up sufficient sample quantity. Then, introduce the stick once into 4 different parts of the stool sample (2), to collect faecal sample (approx. 125mg) and add it to the collection tube. For liquid samples, add approx. 125µL in the collection tube using a micropipette.
2. Close the tube with the diluent and stool sample. Shake the tube in order to assure good sample dispersion (3).

MATERIALS

MATERIALS PROVIDED

- CerTest *Rotavirus-Adenovirus* card tests
- Instructions for use
- Stool collection tubes with diluent

MATERIALS REQUIRED BUT NO PROVIDED

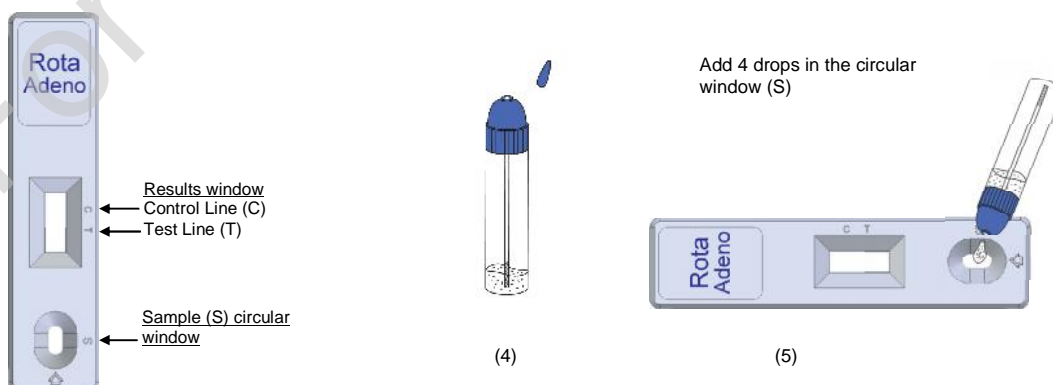
- Specimen collection container
- Disposable gloves
- Timer

TEST PROCEDURE


Allow tests, stool samples and controls to reach room temperature (15-30°C) prior to testing. Do not open pouches until the performance of the assay.

1. Proceed to shake the stool collection tube in order to assure good sample dispersion.
2. Remove CerTest *Rotavirus-Adenovirus* card test from its sealed bag just before using it.
3. Take the stool collection tube, cut the end of the cap (4) and dispense 4 drops in the circular window marked with the letter S (5). Avoid adding solid particles with the liquid.
4. **Read the result at 10 minutes.** Do not read the test result later than 10 minutes.

If the test does not run due to solid particles, stir the sample added in the sample window (S) with the stick. If it doesn't work, dispense a drop of diluent until seeing the liquid running through the reaction zone.

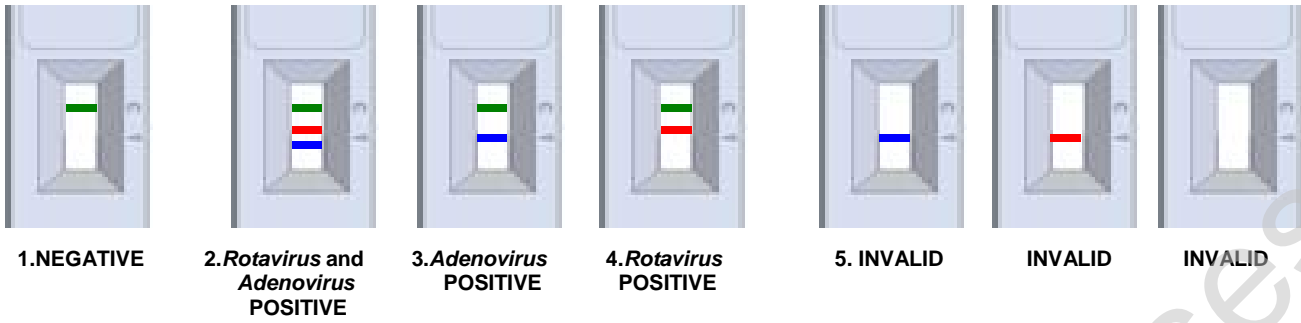INTERPRETATION OF THE RESULTS (please refer to the illustration below)

	Rotavirus-Adenovirus	Interpretation of the results
1.	- GREEN	There is no <i>Rotavirus</i> or <i>Adenovirus</i> presence. No viral infection caused by <i>Rotavirus</i> or <i>Adenovirus</i> .
2.	+/+ GREEN-RED-BLUE	There is <i>Rotavirus</i> and <i>Adenovirus</i> presence. Viral infection caused by <i>Rotavirus</i> and <i>Adenovirus</i> .
3.	-/+ GREEN-BLUE	There is <i>Adenovirus</i> presence. Viral infection caused by <i>Adenovirus</i> .
4.	+/- GREEN-RED	There is <i>Rotavirus</i> presence. Viral infection caused by <i>Rotavirus</i> .
5.	Any other result	Invalid result, we recommend repeating the assay using the same sample with another test.

NEGATIVE: Only the GREEN line appears across the results window in the site marked with the letter C (control line).

Rotavirus and Adenovirus POSITIVE: In addition to the GREEN control line, a RED line and a BLUE line (test lines) also appear in the site marked with the letter T in the window results, during the test performance.

Adenovirus POSITIVE: In addition to the GREEN control line, a BLUE line (*Adenovirus* test line) also appears in the site marked with the letter T (test line) in the results window.

Rotavirus POSITIVE: In addition to the GREEN control line, a RED line (*Rotavirus* test line) also appears in the site marked with the letter T (test line) in the results window.

INVALID: Total absence of the control coloured line (GREEN) regardless the appearance or not of the test lines (RED/BLUE). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persist, discontinue using the test kit and contact your local distributor.

NOTES ON THE INTERPRETATION OF RESULTS

The intensity of the red and/or blue coloured bands in the test lines (T) in the results window will vary depending on the concentration of antigens present in the specimen. However, neither the quantitative value nor the rate of increase in antigens can be determined by this qualitative test.

QUALITY CONTROL

Internal procedural control is included in the test. A green line appearing in the control line (C) in the results window is an internal control, which confirms sufficient specimen volume and correct procedural technique.

LIMITATIONS

1. The test must be carried out within 2 hours after opening the sealed bag.
2. An excess of sample could cause wrong results (brown bands appear). Dilute the sample with the diluent and repeat the test.
3. The intensity of test line may vary from very strong at high antigens concentration to faint when the antigens concentration is close to the detection limit value of the test.
4. After one week of infection, the number of viruses in faeces is decreasing, making the sample less reactive. Stool samples should be collected within one week of the onset of symptoms.

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- 5. CerTest *Rotavirus-Adenovirus* should be used only with samples from human faeces. The use of other samples has not been established. The quality of the test depends on the quality of the sample; proper faecal specimens must be obtained.
- 6. Positive results determine the presence of *Rotavirus* and/or *Adenovirus* in faecal samples; nevertheless, a positive result should be followed up with additional laboratory techniques (biochemical methods or microscopy) to confirm the results. A confirmed infection should only be made by a physician after all clinical and laboratory findings have been evaluated and must be based in the correlation of the results with further clinical observations.
- 7. A negative result is not meaningful because of it is possible the antigens concentration in the stool sample is lower than the detection limit value. If the symptoms or situation still persist, a *Rotavirus* and/or *Adenovirus* determination should be carried out with another technique (for example microscopy).

EXPECTED VALUES

Acute gastroenteritis is a common disorder in young children; moreover, the associated dehydration is a leading cause of admission to hospital in industrialized countries. Acute diarrhea is a major health problem throughout the world and main source of mortality in developing countries. Enteric viruses have been recognized as the most significant etiological agents of the disease and yet, four categories of viruses are being considered clinically relevant: group A *Rotavirus* (family *Reoviridae*), *Norovirus* (family *Caliciviridae*), *Adenovirus* and *Astrovirus*. Several studies proved co-infections in infants 46% with acute watery diarrhea.

PERFORMANCE CHARACTERISTICS

Clinical sensitivity and specificity

An evaluation, with faecal samples, was performed using an immunochromatographic test (CerTest *Rotavirus-Adenovirus*, CerTest) and these results were compared with a commercial available test (Ridascreen®*Rotavirus* ELISA Test, r-Biopharm) for *Rotavirus* line and the results for *Adenovirus* line were confirmed using PCR. The results were as follows:

CerTest <i>Rotavirus-Adenovirus</i> (<i>Rotavirus</i>)	Ridascreen® <i>Rotavirus</i> ELISA Test			
		+	-	Total
	+	18	1	19
-	0	43	43	
Total	18	44	62	

	Sensitivity	Specificity	PPV	NPV
CerTest <i>Rotavirus-Adenovirus</i> (<i>Rotavirus</i>) vs Ridascreen® <i>Rotavirus</i> ELISA Test	>99%	98%	>94%	>99%

CerTest <i>Rotavirus-Adenovirus</i> (<i>Adenovirus</i>)	PCR			
		+	-	Total
	+	7	0	7
-	0	52	52	
Total	7	52	59	

	Sensitivity	Specificity	PPV	NPV
CerTest <i>Rotavirus-Adenovirus</i> (<i>Adenovirus</i>) vs PCR	>99%	>99%	>99%	>99%

The results showed a high sensitivity and specificity to detect *Rotavirus* and *Adenovirus* using CerTest *Rotavirus-Adenovirus*.

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
Cross reactivity

An evaluation was performed to determine the cross reactivity of CerTest *Rotavirus-Adenovirus*; no cross reactivity against gastrointestinal pathogens occasionally present in faeces:
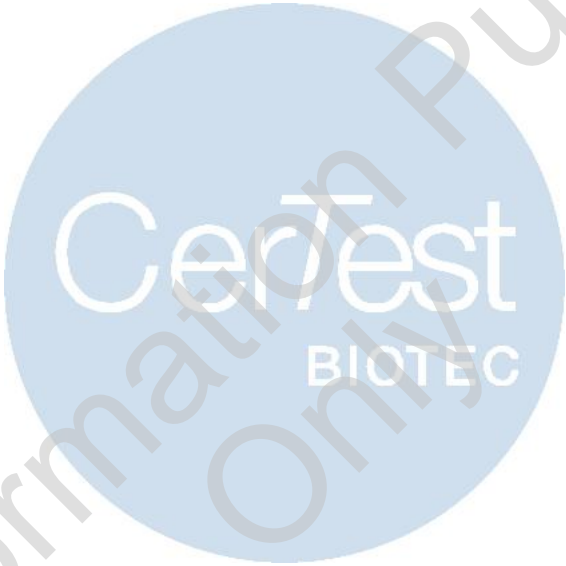
<i>Adenovirus (Rotavirus line)</i>	<i>Escherichia coli O157:H7</i>	<i>Salmonella typhimurium</i>
<i>Astrovirus</i>	<i>Giardia lamblia</i>	<i>Salmonella typhi</i>
<i>Campylobacter coli</i>	<i>Helicobacter pylori</i>	<i>Shigella boydii</i>
<i>Campylobacter jejuni</i>	<i>Listeria monocytogenes</i>	<i>Shigella dysenteriae</i>
<i>Clostridium difficile</i>	<i>Norovirus</i>	<i>Shigella flexneri</i>
<i>Cryptosporidium parvum</i>	<i>Rotavirus (Adenovirus line)</i>	<i>Shigella sonnei</i>
<i>Entamoeba histolytica</i>	<i>Salmonella enteritidis</i>	<i>Staphylococcus aureus</i>
<i>Enterovirus</i>	<i>Salmonella paratyphi</i>	<i>Yersinia enterocolitica</i>

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