# REAL TIME PCR DETECTION KIT

## Astrovirus

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VIASURE <i>Astrovirus</i> Real Time PCR Detection Kit 6 x 8-well strips, low profile	<b>VS-ATV106L</b>
VIASURE <i>Astrovirus</i> Real Time PCR Detection Kit 6 x 8-well strips, high profile	<b>VS-ATV106H</b>
VIASURE <i>Astrovirus</i> Real Time PCR Detection Kit 12 x 8-well strips, low profile	<b>VS-ATV112L</b>
VIASURE <i>Astrovirus</i> Real Time PCR Detection Kit 12 x 8-well strips, high profile	<b>VS-ATV112H</b>
VIASURE <i>Astrovirus</i> Real Time PCR Detection Kit 96-well plate, low profile	<b>VS-ATV113L</b>
VIASURE <i>Astrovirus</i> Real Time PCR Detection Kit 96-well plate, high profile	<b>VS-ATV113H</b>

VIASURE

## ENGLISH

---

### 1. Intended of use

---

VIASURE *Astrovirus* Real Time PCR Detection Kit is designed for specific identification and quantification of Astrovirus in human stool samples from patients with signs and symptoms of gastrointestinal infection. This test is intended for use as an aid in the diagnosis of Astrovirus in humans in combination with clinical and epidemiological risk factors. RNA is extracted from specimens, amplified using RT-amplification and detected using fluorescent reporter dye probes specific for Astrovirus.

### 2. Summary and Explanation

---

Human astroviruses (HAstVs) belong to the *Astroviridae* family, genus *Mamastrovirus* (MAstV). Classic HAstVs contain 8 serotypes and account for 2 to 9% of all acute nonbacterial gastroenteritis in children worldwide, although infections in healthy adults, elderly people and immunocompromised hosts are also reported. The age of children infected with classic HAstV is highly variable, ranging from newborns to over 5 years; however, infection is more common among those younger than 2. Typically, HAstV infection induces a mild, watery diarrhea that lasts 2 to 3 days, associated with vomiting, fever, anorexia, and abdominal pain. The infection is transmitted essentially through the fecal-oral route, even though food and water may act as vehicles for human enteric virus transmission.

The genome of Human Astrovirus consists of a single molecule of positive-sense single-stranded RNA (ssRNA) which contains three open reading frames (ORFs), named ORF1a, ORF1b, and ORF2. ORF1a and ORF1b encode the nonstructural proteins (nsPs) involved in RNA transcription and replication, while ORF2 encodes the structural proteins. The genomic region ORF1b has been shown to be the best target for detection due to its highly conserved sequences along Astrovirus.

### 3. Principle of the procedure

---

VIASURE *Astrovirus* Real Time PCR Detection Kit is designed for the diagnosis of gastroenteritis caused by Astrovirus in human stool samples. The detection is done in one step real time RT format where the reverse transcription and the subsequent amplification of specific target sequence occur in the same reaction well. The isolated RNA target is transcribed generating complementary DNA by reverse transcriptase which is followed by real-time amplification of target sequence of Astrovirus. Identification of Astrovirus is performed by the use of target specific primers and a fluorescent-labeled probe that hybridizes to a conserved region with the genomic region ORF1b.

VIASURE *Astrovirus* Real Time PCR Detection Kit is based on 5' nuclease chemistry. This technology utilizes two primers and a hydrolysis probe and exploits the exonuclease activity of Taq DNA polymerase. During DNA amplification, this enzyme cleaves the probe bound to the complementary DNA sequence, separating the quencher dye from the reporter.

This step generates an increase in fluorescent signal which is proportional to the quantity of target sequence is presents in the sample. This fluorescence could be measured on a range of real time PCR platforms.

VIASURE *Astrovirus* Real Time PCR Detection Kit contains in each well all the components necessary for real time PCR assay (specific primers/probes, dNTPS, buffer, polymerase, retrotranscriptase) in an stabilized format, as well as an internal control to monitor PCR inhibition. *Astrovirus* RNA targets are amplified and detected in FAM channel and the internal control (IC) in HEX channel.

#### 4. Reagents provided

VIASURE *Astrovirus* Real Time PCR Detection Kit includes the following materials and reagents detailed in Table 1 and Table 2:

Reference	Reagent/Material	Description	Color	Amount
VS-ATV1SL/ VS-ATV1SH	<i>Astrovirus</i> 8-well strips	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and Internal control in stabilized format	White	6/12 X 8-well strip
VS-RB02	Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
VS-ATV1C	<i>Astrovirus</i> Positive Control	Non-infectious synthetic lyophilized cDNA	Red	1 vial
VS-NC1	Negative control	Non template control	Violet	1 vial x 1 mL
VS-H2O	Water RNAse/DNAse free	Water RNAse/DNAse free	White	1 vial x 1 mL
VS-OCS	Tear-off 8-cap strips	Optical caps for sealing wells during thermal cycling	Transparent	6/12 X 8-cap strip

Table 1. Reagents and materials provided in VIASURE *Astrovirus* Real Time PCR Detection Kit with Ref. VS-ATV106L, VS-ATV106H, VS-ATV112L and VS-ATV112H.

Reference	Reagent/Material	Description	Color	Amount
VS-ATV1PL/ VS-ATV1PH	<i>Astrovirus</i> 96-well plate	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and Internal control in stabilized format	White	1 plate
VS-RB02	Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
VS-ATV1C	<i>Astrovirus</i> Positive Control	Non-infectious synthetic lyophilized cDNA	Red	1 vial
VS-NC1	Negative control	Non template control	Violet	1 vial x 1 mL
VS-H2O	Water RNAse/DNAse free	Water RNAse/DNAse free	White	1 vial x 1 mL
VS-FILM	Optical Sealing Film	Optical cover film for sealing plate during thermal cycling	Transparent	2 films

Table 2. Reagents and materials provided in VIASURE *Astrovirus* Real Time PCR Detection Kit with Ref. VS-ATV113L and VS-ATV113H.

## 5. Reagents and equipment to be supplied by the user

---

The following list includes materials that are required for using but not included in the VIASURE *Astrovirus* Real Time PCR Detection Kit.

- Real Time PCR instrument (thermocycler) (to check compatibility see Annex I).
- RNA extraction kit.
- Centrifuge for 1.5 mL tubes.
- Vortex.
- Micropipettes (0.5-20  $\mu$ L, 20-200  $\mu$ L).
- Filter tips.
- Powder-free disposal gloves.

## 6. Transport and storage conditions

---

- The kits can be shipped at 2-50°C for a maximum of 2 weeks.
- For long term storage the kit should be stored at 2-8°C.
- Once the positive control has been re-suspended, store it at -20°C. We recommend to separate in aliquots to minimize freeze and thaw cycles.
- Keep components away from sunlight.

## 7. Precautions for users

---

- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- Design a unidirectional process flow. It should begin in the Extraction Area and then move to the Amplification and Detection Area. Do not return samples, equipment and reagents to the area in which the previous step was performed.
- Follow Good Laboratory Practices. Wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area. Once you finish the test wash your hands.
- Specimens must be treated as potentially infectious as well as all reagents and materials that have been exposed to the samples and must be handled according to the national safety regulations. Take necessary precautions during the collection, storage, treatment and disposal of samples.
- Regular decontamination of commonly used equipment is recommended, especially micropipettes and work surfaces.

## 8. Test procedure

### 8.1. SAMPLE PREPARATION

Stool samples should be collected in clean containers and processed as soon as possible to guarantee the quality of the test. We recommend to use fresh samples.

For longer storage, the samples must be frozen at -20°C. In this case, the sample will be totally thawed and brought to room temperature before testing. Homogenise stool sample as thoroughly as possible prior to preparation. Freezing and thawing cycles are not recommended.

We recommend to dilute stool samples before extraction. Collect a pea-size stool (approx. 8mm) and put in a 1.5 mL microcentrifuge tube containing 100 µL of physiological serum or PBS. Vortex intensely and centrifuge 10,000 rpm for 1min. Use 200 µL of supernatant to perform RNA extraction.

#### 8.1.1. RNA EXTRACTION

For RNA extraction from human stool samples you can use your manually or automatic routine optimized system. Also, you can use any commercially available RNA extraction kit and follow the manufacturer's instructions for use. We have validated the following extraction kits:

- Viasure RNA-DNA Extraction kit (VIASURE), recommended.
- Invisorb® Spin Universal Kit (Stratec).
- UltraClean® Tissue & Cells DNA/RNA Isolation Kit (Mobio).
- NucleoSpin® RNA Virus (Machery Nagel).

### 8.2. LYOPHILIZED POSITIVE CONTROL

*Astrovirus* Positive Control contains high copies template the recommendation is to open and manipulate it in a separate laboratory area away from the other components. Reconstitute the lyophilized *Astrovirus* Positive Control (red vial) adding 100 µL of Water RNase/DNase free (with vial) supplied and vortex thoroughly.

Once the positive control has been re-suspended, store it at -20°C. We recommend to separate in aliquots to minimize freeze and thaw cycles.

### 8.3. PCR PROTOCOL

Determine and separate the number of required reactions including samples and controls. One positive and negative control must be included in each run. Peel off protective aluminum seal from plates or strips.

#### 1) Reconstitute the number of wells you need.

Add 15 µL of Rehydration Buffer (blue vial) into each well.

2) Adding samples and controls.

Add 5 µL of RNA sample, reconstituted *Astrovirus* Positive Control (red vial) or Negative Control (violet vial) in different wells and close the wells with the caps provided. Centrifuge briefly.

Load the plate or the strips in the thermocycler.

3) Set up your thermocycler.

Program your thermocycler following the conditions below and start the run:

Cycles	Step	Time	Temperature
1	Reverse transcription	15 min	45°C
1	Initial denaturalization	2 min	95°C
45	Denaturalization	10 seg	95°C
	Annealing/Data collection*	50 seg	60°C

Table 3. PCR protocol

Fluorogenic data should be collected during the annealing step (\*) through the FAM (*Astrovirus*) and HEX, JOE or VIC channels (Internal Control (IC)). In Applied Biosystems 7500 Fast Real-Time PCR System, Applied Biosystems StepOne™ Real-Time PCR System, and Stratagene Mx3005P™ Real Time PCR System check that passive reference option ROX is none.

## 9. Result interpretation

The use of positive and negative controls in each run, validate the reaction by checking the absence of signal in negative control well and the presence of signal for *Astrovirus* positive control well. Check Internal Control signal to verify the correct functioning of the amplification mix. The analysis of the samples is done by the software itself of the used real time PCR equipment according to manufacturer’s instructions.

Using the following table read and analyze the results:

Astrovirus	Internal control	Negative control	Positive control	Interpretation
+	+/-	-	+	<b>Astrovirus Positive</b>
-	+	-	+	<b>Astrovirus Negative</b>
+	+	+	+	<b>Experiment fail</b>
-	-	-	-	<b>Experiment fail</b>

Table 4. Sample interpretation


+: Amplification curve  
 -: No amplification curve

A sample is considered positive if the Ct value obtained is less than 40 and the internal control shows an amplification signal.

A sample is considered positive if the sample shows an amplification signal less than 40 Ct value but the internal control is negative. Sometimes, the detection of internal control is not necessary because a high copy number of target can cause preferential amplification of target-specific nucleic acids.

A sample is considered negative, if the sample shows no amplification signal in the detection system but the internal control is positive. An inhibition of the PCR reaction can be excluded by the amplification of internal control.

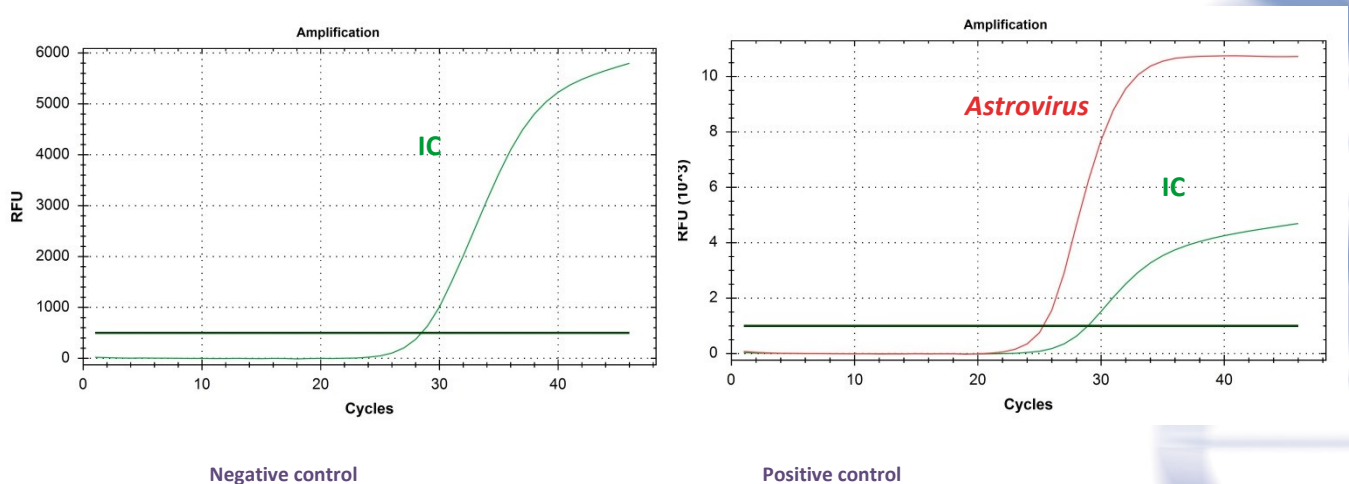Figure 1. Correct run of negative and positive control run on the Bio-Rad CFX96 Touch™ Real-Time PCR Detection System.

The result is considered invalid if there is signal of amplification in negative control or absence of signal in the positive well. We recommend to repeat the assay again.

In case of absence of internal control signal in sample wells we recommend to repeat the assay diluting the sample 1:10 or to repeat the extraction to check for possible problems of inhibition.

## 10. Limitations of the test

- The result of the test should be evaluated by a health care professional and evaluated in the context of medical history, clinical symptoms and other diagnostic tests.
- Although this assay can be used with other types of samples it has been validated with human faecal samples.
- The quality of the test depends on the quality of the sample; proper RNA from faecal specimens must be extracted. Unsuitable collection, storage and/or transport of specimens may give false negative results.
- Extremely low levels of target below the limit of detection may be detected, but results may not be reproducible.
- There is a possibility of false positive results due to cross-contamination by Astrovirus, either samples containing high concentrations of target RNA or contamination due to PCR products from previous reactions.

## 11. Quality control

VIASURE *Astrovirus* Real Time PCR Detection Kit contains a positive and a negative control that must be included in each run to correctly interpret the results. Also, the internal control (IC) in each well confirms the correct performance of the technique.

## 12. Performance characteristics

### 12.1. CLINICAL SENSITIVITY AND SPECIFICITY

Evaluation with faecal samples from symptomatic patients was tested using VIASURE *Astrovirus* Real Time PCR Detection Kit. These results were compared with those obtained by a commercial Real Time PCR Kit (RIDA®GENE Viral Stool Panel II (r-Biopharm)).

The results were as follows:

VIASURE <i>Astrovirus</i> Real Time PCR Detection Kit	RIDA®GENE Viral Stool Panel II (r-Biopharm)			Total
		+	-	
+	30	1 <sup>#*</sup>	31	
-	1 <sup>#*</sup>	40	41	
Total	31	41	72	

Table 5. Comparative results

*\*The low amount of template RNA detected in these faecal samples is below the detection limit of the method used and so it may produce random positive results.*

*#Besides, these two samples have been evaluated by an additional commercial Real Time RT-PCR Kit (Rotavirus/Norovirus/Astrovirus Real-TM (Sacace)), confirming our results.*


The results show a high sensitivity and specificity to detect *Astrovirus* using VIASURE *Astrovirus* Real Time PCR Detection Kit.

### 12.2. ANALYTICAL SENSITIVITY

VIASURE *Astrovirus* Real Time PCR Detection Kit has a detection limit of  $\geq 100$  viral RNA copies per reaction (Figure 2)

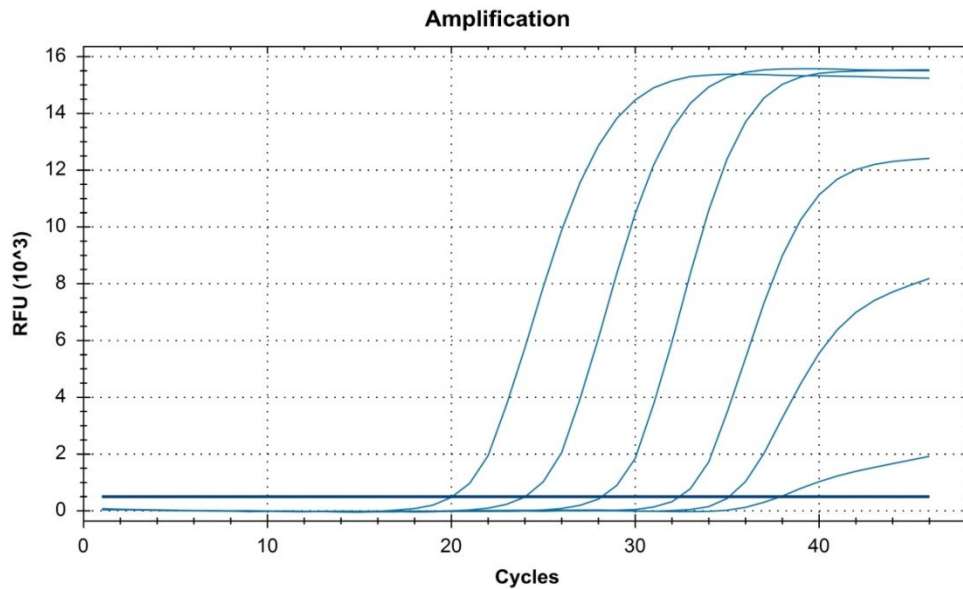Figure 2. Dilution series of Astrovirus ( $10^7$ - $10^2$  copies/rxn) template run on the Bio-Rad CFX96 Touch™ Real-Time PCR Detection System.

**12.3. ANALYTICAL SPECIFICITY**

The specificity of the Astrovirus assay was confirmed by testing a panel consisting of 45 microorganisms representing the most common enteric pathogens or flora present in the intestine.

VIASURE Astrovirus Real Time PCR Detection Kit not identify the following strains confirming the exclusivity of the trial.

Cross-reactivity testing					
<i>Helicobacter pylori</i>	-	<i>Campylobacter lari</i>	-	<i>Klebsiella oxytoca</i>	-
<i>Helicobacter hepaticus</i>	-	<i>Campylobacter fetus</i>	-	<i>Listeria monocytogenes</i>	-
<i>Helicobacter cinaedi</i>	-	<i>Campylobacter coli</i>	-	<i>Candida albicans</i>	-
<i>Helicobacter heilmannii</i>	-	<i>Campylobacter jejuni subsp. jejuni</i>	-	<i>Arcobacter butzleri</i>	-
<i>Shigella flexneri</i>	-	<i>Campylobacter upsaliensis</i>	-	<i>Pseudomonas aeruginosa</i>	-
<i>Shigella dysenteriae</i>	-	<i>Proteus vulgaris</i>	-	<i>Enterococcus faecalis</i>	-
<i>Salmonella typhi</i>	-	<i>Aeromonas hydrophila subsp. hydrophila</i>	-	<i>Giardia intestinalis</i>	-
<i>Salmonella paratyphi A</i>	-	<i>Citrobacter freundii</i>	-	<i>Yersinia enterocolitica O:3</i>	-
<i>Salmonella paratyphi B</i>	-	<i>Staphylococcus aureus subsp. aureus</i>	-	<i>Yersinia enterocolitica O:9</i>	-
<i>Salmonella typhimurium</i>	-	<i>Serratia liquefaciens</i>	-	<i>Bacteroides fragilis</i>	-
<i>Salmonella bongori</i>	-	<i>Vibrio parahaemolyticus</i>	-	<i>Cryptosporidium parvum</i>	-
<i>Salmonella enteritidis</i>	-	<i>Clostridium difficile</i>	-	<i>Entamoeba histolytica</i>	-
<i>Salmonella enterica subsp. entérica</i>	-	<i>Clostridium perfringens</i>	-	Adenovirus serotypes 40/41	-
<i>Salmonella pullorum</i>	-	<i>Enterotoxigenic Escherichia coli</i>	-	Rotavirus A	-
<i>Salmonella gallinarum</i>	-	<i>Enteropathogenic Escherichia coli</i>	-	Norovirus Genotypes I and II	-

Table 6. Reference pathogenic and non-pathogenic microorganisms used in this study.

**12.4. ANALYTICAL REACTIVITY**

The reactivity of VIASURE Astrovirus Real Time PCR Detection Kit was evaluated against human Astrovirus serotypes I-VIII showing positive results.

**ANNEX 1:**

**COMPATIBILITY OF THE MOST COMMON REAL TIME PCR EQUIPMENT**

Low profile strips can be used in all PCR thermocyclers equipped with low profile block, like systems listed in table A.1. High profile strips can be used in all PCR thermocyclers equipped with high or regular profile block, like systems listed in table A.2. If you do not find your thermocycler in the list below, please contact with your supplier.

<b>Table A.1 LOW PROFILE BLOCK THERMOCYCLERS</b>	
<b>Manufacturer</b>	<b>Model</b>
Applied Biosystems	7500 Fast Real-Time PCR System
Applied Biosystems	7500 Fast Dx Real-Time PCR System
Applied Biosystems	QuantStudio™ 12K Flex 96-well Fast
Applied Biosystems	QuantStudio™ 6 Flex 96-well Fast
Applied Biosystems	QuantStudio™ 7 Flex 96-well Fast
Applied Biosystems	StepOne Plus™ Real-Time PCR System
Applied Biosystems	StepOne™ Real-Time PCR System
Applied Biosystems	ViiA™ 7 Fast Real-Time PCR System
Bio-Rad	CFX96 Touch™ Real-Time PCR Detection System
Bio-Rad	Mini Opticon™ Real-Time PCR Detection System
Roche	LightCycler®480 Real-Time PCR System
Roche	LightCycler®96 Real-Time PCR System
Agilent Technologies	AriaMx Real-time PCR System

<b>Table A.2 HIGH PROFILE BLOCK THERMOCYCLERS</b>	
<b>Manufacturer</b>	<b>Model</b>
Applied Biosystems	7300 Real-Time PCR System
Applied Biosystems	7500 Real-Time PCR System
Applied Biosystems	7900 HT Real-Time PCR System
Applied Biosystems	ABI PRISM 7000
Applied Biosystems	ABI PRISM 7700
Applied Biosystems	QuantStudio™ 12K Flex 96-well
Applied Biosystems	QuantStudio™ 6 Flex 96-well
Applied Biosystems	QuantStudio™ 7 Flex 96-well
Applied Biosystems	ViiA™ 7 Real-Time PCR System
Bio-Rad	CFX96 Touch™ Deep Well Real-Time PCR Detection System
Bio-Rad	iCycler iQ™ Real-Time PCR Detection System
Bio-Rad	iCycler iQ™5 Real-Time PCR Detection System
Bio-Rad	MyiQ™ Real-Time PCR Detection System
Bio-Rad	MyiQ™2 Real-Time PCR Detection System
Eppendorf	Mastercycler™ep <i>realplex</i>
Stratagene / Agilent Technologies	Mx3000P™ Real Time PCR System
Stratagene / Agilent Technologies	Mx3005P™ Real Time PCR System
Analytik Jena Biometra	TOptical
Analytik Jena Biometra	qTOWER 2.0

Table A1/A2. Compatible low and high profile Real Time PCR systems.

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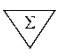
## 13. Bibliography/Bibliografía

1. S. Guix et al. Human astrovirus diagnosis and typing: current and future prospects. *Letters in Applied Microbiology* 2005; 41(2): 103-105.
2. D. Sano et al. Detection of oxidative damages on viral capsid protein for evaluating structural integrity and infectivity of human norovirus. *Environmental Science & Technology* 2010; 44(2): 808-812.
3. A. Bosch et al. Human Astroviruses. *Clinical Microbiology Reviews* 2014; 27(4): 1048-1074.

## 14. Symbols for IVD components and reagents/Símbolos para reactivos y productos para diagnóstico *in vitro*

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
VIASURE *Astrovirus* Real Time PCR Detection Kit has been validated on the following equipments: Applied Biosystems 7500 Fast Real-Time PCR System, Applied Biosystems StepOne™ Real-Time PCR System, Bio-Rad CFX96 Touch™ Real-Time PCR Detection System and AriaMx Real-time PCR System (Agilent Technologies). When using the Applied Biosystems 7500 Fast with strips it is recommend to place a plate holder to reduce the risk of crushed tube (Ref. PN 4388506).

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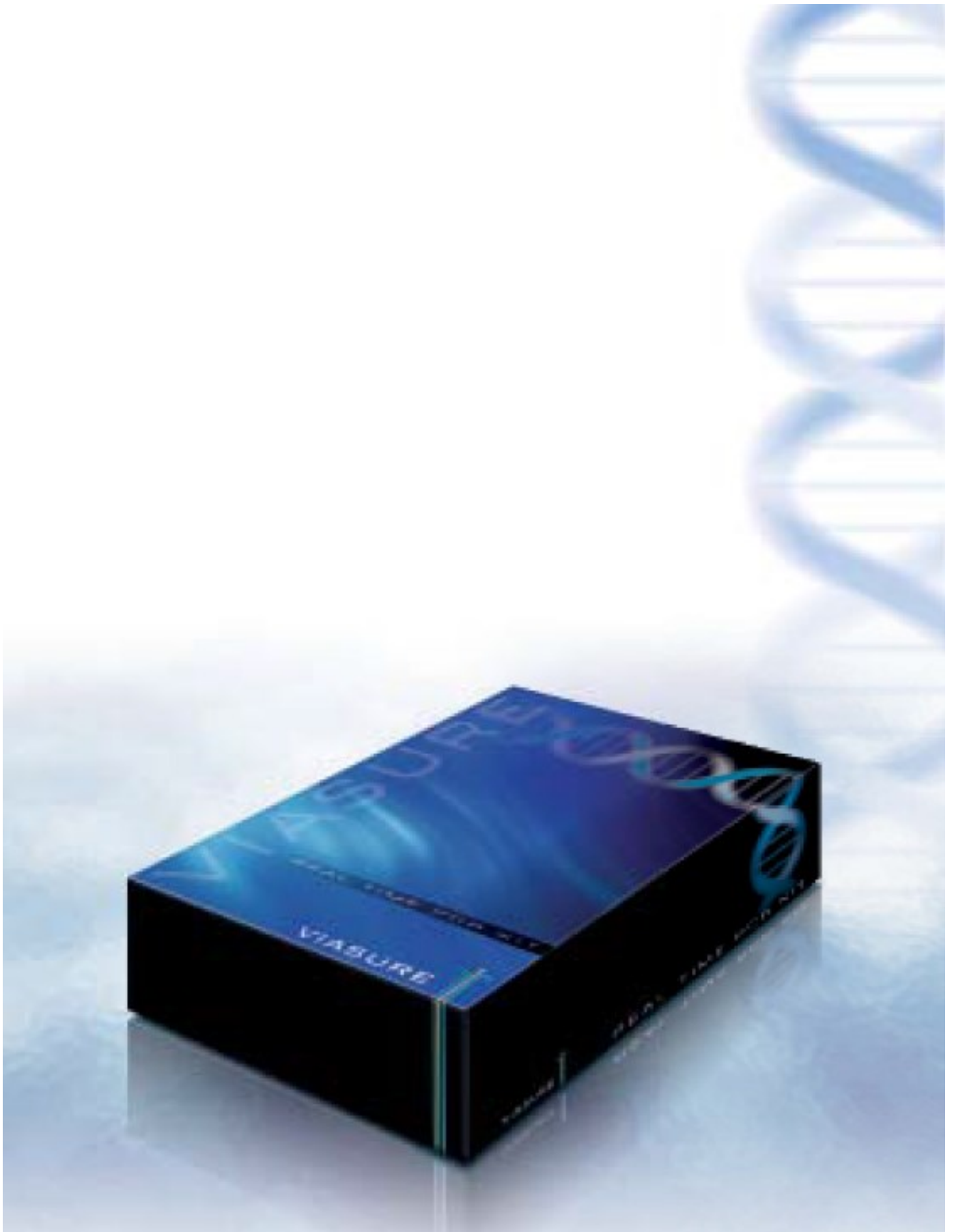- CFX™ and IQ5™ are registered trademarks of Bio-Rad Laboratories
- ABI®, QuantStudio™, StepOnePlus™ and ViiA™ are registered trademarks of Thermo Fisher Scientific Inc.
- LightCycler® is a registered trademark of Roche
- Mx3000P™, Mx3005™ and AriaMx are registered trademarks of Agilent Technologies
- Mastercycler™ is a registered trademark of Eppendorf.

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