REAL TIME PCR DETECTION KIT

Helicobacter pylori

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VIASURE <i>Helicobacter pylori</i> Real Time PCR Detection Kit 6 x 8-well strips, low profile	VS-PYR106L
VIASURE <i>Helicobacter pylori</i> Real Time PCR Detection Kit 6 x 8-well strips, high profile	VS-PYR106H
VIASURE <i>Helicobacter pylori</i> Real Time PCR Detection Kit 12 x 8-well strips, low profile	VS-PYR112L
VIASURE <i>Helicobacter pylori</i> Real Time PCR Detection Kit 12 x 8-well strips, high profile	VS-PYR112H
VIASURE <i>Helicobacter pylori</i> Real Time PCR Detection Kit 96-well plate, low profile	VS-PYR113L
VIASURE <i>Helicobacter pylori</i> Real Time PCR Detection Kit 96-well plate, high profile	VS-PYR113H

VIASURE

ENGLISH

1. Intended of use

VIASURE *Helicobacter pylori* Real Time PCR Detection Kit is designed for specific identification of *Helicobacter pylori* in human stool samples from patients with signs and symptoms of *Helicobacter pylori* infection. This test is intended for use as an aid in the diagnosis of *Helicobacter pylori* infection in humans in combination with clinical and epidemiological risk factors. DNA is extracted from stool specimens, multiplied using Real Time amplification and detected using specific primers and a fluorescent reporter dye probe for *Helicobacter pylori*.

2. Summary and Explanation

The genus *Helicobacter* belongs to the family *Helicobacteriaceae* of the order *Campylobacterales*. *Helicobacter pylori* (*H. pylori*) is a gram-negative microaerophilic spiral-shaped bacterium, which is able to colonize the mucus layer of the human stomach and the upper part of small intestine (duodenum).

More than half of the world population is estimated to be infected with *H. pylori*, but most individuals are asymptomatic. The new infections might be due to direct person-to-person transmission, via either an oral-oral, fecal-oral route or both. Nevertheless, the role of coccoid form of *H. pylori* as a vehicle of infection from food and water sources should not be discarded. *H. pylori* is involved in the pathogenesis of atrophic gastritis, gastroduodenal ulcer, gastric cancer and gastric mucosa-associated lymphoid tissue (MALT) lymphoma.

At present, several diagnostic assays for *H. pylori* detection are available and grouped as “invasive” or “noninvasive”, but none of them can be considered as gold standard alone. Invasive methods include histology, culture and rapid urease testing, which require gastric biopsy specimens obtained by gastroduodenoscopy. Noninvasive approaches include fecal antigen detection, serologic testing, and urea breath testing among others. Urease is an important factor for the maintenance and virulence of the bacterium in the gastric mucosa. It is composed of two structural subunits encoded by genes, *ureA* and *ureB*, which have been frequently employed as target genes for the specific detection of *Helicobacter pylori*.

3. Principle of the procedure

VIASURE *Helicobacter pylori* Real Time PCR Detection Kit is designed for the diagnosis of *Helicobacter pylori* in human stool samples. After DNA isolation, the identification of *Helicobacter pylori* is performed by the amplification of a conserved region with the *ureB* gene using specific primers and a fluorescent-labeled probe.

VIASURE *Helicobacter pylori* Real Time PCR Detection Kit is based on 5' exonuclease activity of DNA polymerase. During DNA amplification, this enzyme cleaves the probe bound to the complementary DNA sequence, separating the quencher dye from the reporter. This reaction generates an increase in the fluorescent signal which is proportional to the quantity of the hydrolyzed target sequence. This fluorescence could be measured on real time PCR platforms.

VIASURE *Helicobacter pylori* Real Time PCR Detection Kit contains in each well all the components necessary for real time PCR assay (specific primers/probes, dNTPS, buffer, polymerase) in a stabilized format, as well as an internal control to discard the inhibition of the polymerase activity. *Helicobacter pylori* DNA targets are amplified and detected in FAM channel and the internal control (IC) in HEX, VIC or JOE channel (depending on the equipment used).

4. Reagents provided

VIASURE *Helicobacter pylori* Real Time PCR Detection Kit includes the following materials and reagents detailed in Table 1 and Table 2:

Reference	Reagent/Material	Description	Color	Amount
VS-PYR1SL/ VS-PYR1SH	<i>Helicobacter pylori</i> 8-well strips	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and Internal control in stabilized format	White	6/12 X 8-well strip
VS-RB02	Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
VS-PYR1C	<i>Helicobacter pylori</i> Positive Control	Non-infectious synthetic lyophilized DNA	Red	1 vial
VS-NC1	Negative control	Non template control	Violet	1 vial x 1 mL
VS-H2O	Water RNAse/DNAse free	Water RNAse/DNAse free	White	1 vial x 1 mL
VS-OCS	Tear-off 8-cap strips	Optical caps for sealing wells during thermal cycling	Transparent	6/12 X 8-cap strip

Table 1. Reagents and materials provided in VIASURE *Helicobacter pylori* Real Time PCR Detection Kit with Ref. VS-PYR106L, VS-PYR106H, VS-PYR112L and VS-PYR112H.

Reference	Reagent/Material	Description	Color	Amount
VS-PYR1PL/ VS-PYR1PH	<i>Helicobacter pylori</i> 96-well plate	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and Internal control in stabilized format	White	1 plate
VS-RB02	Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
VS-PYR1C	<i>Helicobacter pylori</i> Positive Control	Non-infectious synthetic lyophilized DNA	Red	1 vial
VS-NC1	Negative control	Non template control	Violet	1 vial x 1 mL
VS-H2O	Water RNAse/DNAse free	Water RNAse/DNAse free	White	1 vial x 1 mL
VS-OCS	Tear-off 8-cap strips	Optical caps for sealing plate during thermal cycling	Transparent	12 X 8-cap strip

Table 2. Reagents and materials provided in VIASURE *Helicobacter pylori* Real Time PCR Detection Kit with Ref. VS-PYR113L and VS-PYR113H.

5. Reagents and equipment to be supplied by the user

The following list includes materials that are required for using, but not included in the VIASURE *Helicobacter pylori* Real Time PCR Detection Kit.

- Real Time PCR instrument (thermocycler) (to check compatibility see Annex I).
- DNA extraction kit.
- Centrifuge for 1.5 mL tubes.
- Vortex.
- Micropipettes (0.5-20 µL, 20-200 µL).
- Filter tips.
- Powder-free disposal gloves.

6. Transport and storage conditions

- The kits can be shipped at 2-50°C for a maximum of 2 weeks.
- For long term storage the kit should be stored at 2-8°C.
- Once the positive control has been re-suspended, store it at -20°C. We recommend to separate in aliquots to minimize freeze and thaw cycles.
- Keep components away from sunlight.

7. Precautions for users

- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- Design a unidirectional process flow. It should begin in the Extraction Area and then move to the Amplification and Detection Area. Do not return samples, equipment and reagents to the area in which the previous step was performed.
- Follow Good Laboratory Practices. Wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area. Once you finish the test wash your hands.
- Specimens must be treated as potentially infectious as well as all reagents and materials that have been exposed to the samples and must be handled according to the national safety regulations. Take necessary precautions during the collection, storage, treatment and disposal of samples.
- Regular decontamination of commonly used equipment is recommended, especially micropipettes and work surfaces.

8. Test procedure

8.1. SAMPLE PREPARATION

Stool samples should be collected in clean containers and processed as soon as possible to guarantee the quality of the test. We recommend to use fresh samples.

For longer storage, the samples must be frozen at -20°C. In this case, the sample will be totally thawed and brought to room temperature before testing. Homogenise stool sample as thoroughly as possible prior to preparation. Freezing and thawing cycles are not recommended.

We recommend to dilute stool samples before extraction. Collect a pea-size stool (approx. 8mm) and put in a 1.5 mL microcentrifuge tube containing 100 µL of physiological serum or PBS. Vortex intensely and centrifuge 10,000 rpm for 1min. Use 200 µL of supernatant to perform DNA extraction.

8.1.1. DNA EXTRACTION

For DNA extraction from human stool samples you can use your manually or automatic routine optimized system. Also, you can use any commercially available DNA extraction kit and follow the manufacturer's instructions for use. We have validated the following extraction kits:

- Viasure RNA-DNA Extraction kit (VIASURE), recommended*.
- Invisorb[®] Spin Universal Kit (Stratec).
- UltraClean[®] Tissue & Cells DNA/RNA Isolation Kit (Mobio).
- NucleoSpin[®] RNA Virus (Machery Nagel).

**In order to improve the yield and quality of bacterial DNA from stool samples, we recommend the sample pretreatment with Lysozyme at 37°C as described on "Instructions for use" for bacterial DNA isolation. In addition, usage of small elution volumes may raise the DNA/RNA concentration.*

8.2. LYOPHILIZED POSITIVE CONTROL

Helicobacter pylori Positive Control contains high copies template, the recommendation is to open and manipulate it in a separate laboratory area away from the other components. Reconstitute the lyophilized *Helicobacter pylori* Positive Control (red vial) adding 100 µL of Water RNase/DNase free (white vial) supplied and vortex thoroughly.

Once the positive control has been re-suspended, store it at -20°C. Recommendation is to separate it in aliquots to minimize freeze and thaw cycles.

8.3. PCR PROTOCOL

Determine and separate the number of required reactions including samples and controls. One positive and negative control must be included in each run. Peel off protective aluminum seal from plates or strips.

1) Reconstitute the number of wells you need.

Add 15 µL of Rehydration Buffer (blue vial) into each well.

2) Adding samples and controls.

Add 5 µL of DNA sample, reconstituted *Helicobacter pylori* Positive Control (red vial) or Negative Control (violet vial) in different wells and close the wells with the caps provided. Centrifuge briefly.

Load the plate or the strips in the thermocycler.

3) Set up your thermocycler.

Program your thermocycler following the conditions below and start the run:

Cycles	Step	Time	Temperature
1	Polymerase activation	2 min	95°C
45	Denaturalization	10 seg	95°C
	Annealing/Data collection*	50 seg	60°C

Table 3. PCR protocol.

Fluorogenic data should be collected during the annealing step (*) through the FAM (*Helicobacter pylori*) and HEX, JOE or VIC channels (Internal Control (IC)). In Applied Biosystems 7500 Fast Real-Time PCR System, Applied Biosystems StepOne™ Real-Time PCR System and Stratagene Mx3005P™ Real Time PCR System check that passive reference option ROX is none.

9. Result interpretation

The use of positive and negative controls in each run, validates the reaction by checking the absence of signal in negative control well and the presence of signal for *Helicobacter pylori* positive control well. Check Internal Control signal to verify the correct functioning of the amplification mix. The analysis of the samples is done by the software itself of the used real time PCR equipment according to manufacturer’s instructions for use.

Using the following table, read and analyze the results:

<i>Helicobacter pylori</i>	Internal control	Negative control	Positive control	Interpretation
+	+/-	-	+	<i>Helicobacter pylori</i> Positive
-	+	-	+	<i>Helicobacter pylori</i> Negative
+	+	+	+	Experiment fail
-	-	-	-	Experiment fail

Table 4. Sample interpretation


+: Amplification curve
 -: No amplification curve

A sample is considered positive if the Ct value obtained is less than 40 and the internal control shows an amplification signal.

A sample is considered positive if the sample shows an amplification signal less than 40 Ct value but the internal control is negative. Sometimes, the detection of internal control is not necessary because a high copy number of target can cause preferential amplification of target-specific nucleic acids.

A sample is considered negative, if the sample shows no amplification signal in the detection system but the internal control is positive. An inhibition of the PCR reaction can be excluded by the amplification of internal control.

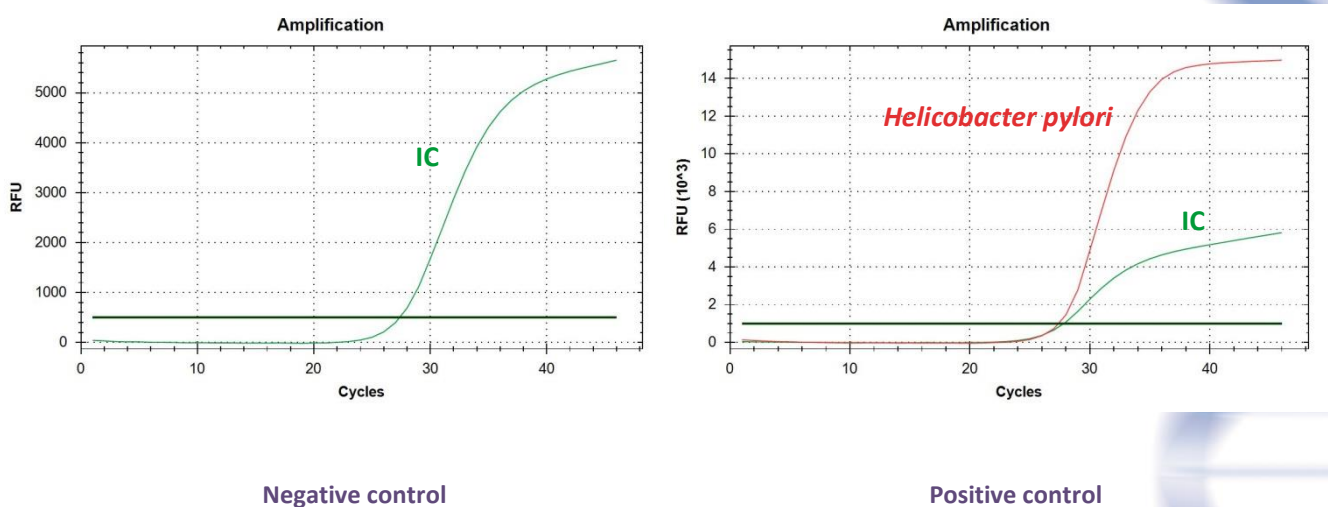Figure 1. Correct run of negative and positive control run on the Bio-Rad CFX96 Touch™ Real-Time PCR Detection System.

The result is considered invalid if there is signal of amplification in negative control or absence of signal in the positive well. We recommend to repeat the assay again.

In case of absence of internal control signal in sample wells we recommend to repeat the assay diluting the sample 1:10 or to repeat the extraction to check for possible problems of inhibition.

10. Limitations of the test

- The result of the test should be evaluated by a health care professional and evaluated in the context of medical history, clinical symptoms and other diagnostic tests.
- Although this assay can be used with other types of samples it has been validated with human faecal samples.
- The quality of the test depends on the quality of the sample; proper DNA from faecal specimens must be extracted. Unsuitable collection, storage and/or transport of specimens may give false negative results.
- Extremely low levels of target below the limit of detection may be detected, but results may not be reproducible.
- There is a possibility of false positive results due to cross-contamination by *Helicobacter pylori*, either samples containing high concentrations of target DNA or contamination due to PCR products from previous reactions.

11. Quality control

VIASURE *Helicobacter pylori* Real Time PCR Detection Kit contains a positive and a negative control that must be included in each run to correctly interpret the results. Also, the internal control (IC) in each well confirms the correct performance of the technique.

12. Performance characteristics

12.1. CLINICAL SENSITIVITY AND SPECIFICITY

Evaluation with faecal samples from symptomatic patients was tested using VIASURE *Helicobacter pylori* Real Time PCR Detection Kit. These results were compared with those obtained by a commercial Real Time PCR Kit (*H. pylori* Real-TM (Sacace)).

The results were as follows:

VIASURE <i>Helicobacter pylori</i> Real Time PCR Detection Kit	<i>H. pylori</i> Real-TM (Sacace)			Total
		+	-	
+	25	1	26	
-	2	14	16	
Total	27	15	42	

Table 5. Comparative results


The results show a high sensitivity and specificity to detect *Helicobacter pylori* using VIASURE *Helicobacter pylori* Real Time PCR Detection Kit.

12.2. ANALYTICAL SENSITIVITY

VIASURE *Helicobacter pylori* Real Time PCR Detection Kit has a detection limit of ≥ 1000 DNA copies per reaction (Figure 2).

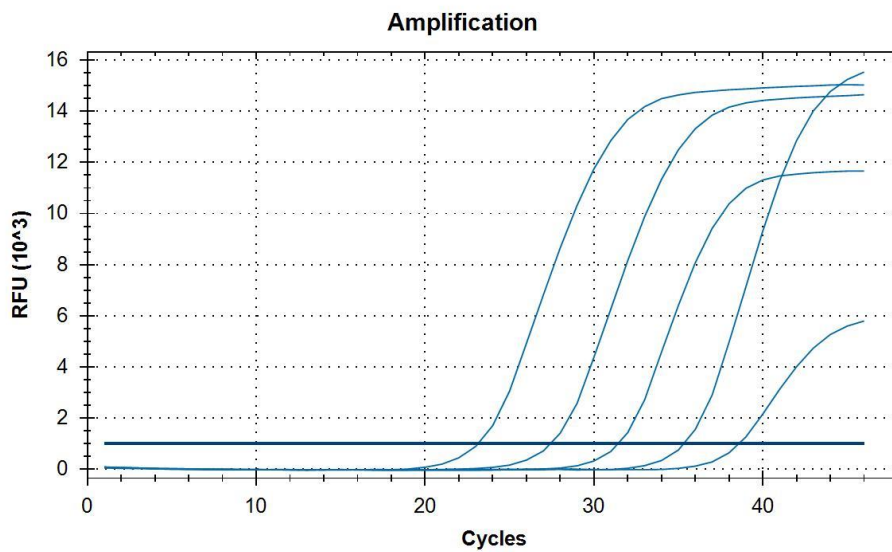Figure 2. Dilution series of *Helicobacter pylori* (10^7 - 10^3 copies/rxn) template run on the Bio-Rad CFX96 Touch™ Real-Time PCR Detection System

12.3. ANALYTICAL SPECIFICITY

The specificity of the *Helicobacter pylori* assay was confirmed by testing a panel consisting of 45 microorganisms representing the most common enteric pathogens or flora present in the intestine.

VIASURE *Helicobacter pylori* Real Time PCR Detection Kit not identify the following 41 strains confirming the exclusivity of the trial. Nevertheless, this kit show cross-reactivity with *H. heilmannii*, which together with *H. pylori* can cause chronic gastritis in humans.

Cross-reactivity testing					
<i>Shigella flexneri</i>	-	<i>Campylobacter upsaliensis</i>	-	<i>Pseudomonas aeruginosa</i>	-
<i>Shigella dysenteriae</i>	-	<i>Proteus vulgaris</i>	-	<i>Enterococcus faecalis</i>	-
<i>Salmonella typhi</i>	-	<i>Aeromonas hydrophila subsp. hydrophila</i>	-	<i>Giardia intestinalis</i>	-
<i>Salmonella paratyphi A</i>	-	<i>Citrobacter freundii</i>	-	<i>Yersinia enterocolitica</i> O:3	-
<i>Salmonella paratyphi B</i>	-	<i>Staphylococcus aureus subsp. aureus</i>	-	<i>Yersinia enterocolitica</i> O:9	-
<i>Salmonella typhimurium</i>	-	<i>Serratia liquefaciens</i>	-	<i>Bacteroides fragilis</i>	-
<i>Salmonella bongori</i>	-	<i>Vibrio parahaemolyticus</i>	-	<i>Cryptosporidium parvum</i>	-
<i>Salmonella enteritidis</i>	-	<i>Clostridium difficile</i>	-	<i>Entamoeba histolytica</i>	-
<i>Salmonella enterica subsp. entérica</i>	-	<i>Clostridium perfringens</i>	-	Adenovirus serotypes 40/41	-
<i>Salmonella pullorum</i>	-	<i>Enterotoxigenic Escherichia coli</i>	-	Rotavirus A	-
<i>Salmonella gallinarum</i>	-	<i>Enteropathogenic Escherichia coli</i>	-	Norovirus Genotypes I and II	-
<i>Campylobacter lari</i>	-	<i>Klebsiella oxytoca</i>	-	Astrovirus Genotype I-VIII	-
<i>Campylobacter fetus</i>	-	<i>Listeria monocytogenes</i>	-	<i>Helicobacter hepaticus</i>	-
<i>Campylobacter coli</i>	-	<i>Candida albicans</i>	-	<i>Helicobacter cinaedi</i>	-
<i>Campylobacter jejuni subsp. jejuni</i>	-	<i>Arcobacter butzleri</i>	-	<i>Helicobacter heilmannii</i>	+

Table 6. Reference pathogenic and non-pathogenic microorganisms used in this study.

12.4. ANALYTICAL REACTIVITY

The reactivity of VIASURE *Helicobacter pylori* Real Time PCR Detection Kit was evaluated against *Helicobacter pylori* strain J99 showing positive result.

ANNEX 1:

COMPATIBILITY OF THE MOST COMMON REAL TIME PCR EQUIPMENT

Low profile strips can be used in all PCR thermocyclers equipped with low profile block, like systems listed in table A.1. High profile strips can be used in all PCR thermocyclers equipped with high or regular profile block, like systems listed in table A.2. If you do not find your thermocycler in the list below, please contact with your supplier.

Table A.1 LOW PROFILE BLOCK THERMOCYCLERS		Table A.2 HIGH PROFILE BLOCK THERMOCYCLERS	
Manufacturer	Model	Manufacturer	Model
Applied Biosystems	7500 Fast Real-Time PCR System	Applied Biosystems	7300 Real-Time PCR System
Applied Biosystems	7500 Fast Dx Real-Time PCR System	Applied Biosystems	7500 Real-Time PCR System
Applied Biosystems	QuantStudio™ 12K Flex 96-well Fast	Applied Biosystems	7900 HT Real-Time PCR System
Applied Biosystems	QuantStudio™ 6 Flex 96-well Fast	Applied Biosystems	ABI PRISM 7000
Applied Biosystems	QuantStudio™ 7 Flex 96-well Fast	Applied Biosystems	ABI PRISM 7700
Applied Biosystems	StepOne Plus™ Real-Time PCR System	Applied Biosystems	QuantStudio™ 12K Flex 96-well
Applied Biosystems	StepOne™ Real-Time PCR System	Applied Biosystems	QuantStudio™ 6 Flex 96-well
Applied Biosystems	ViiA™ 7 Fast Real-Time PCR System	Applied Biosystems	QuantStudio™ 7 Flex 96-well
Bio-Rad	CFX96 Touch™ Real-Time PCR Detection System	Applied Biosystems	ViiA™ 7 Real-Time PCR System
Bio-Rad	Mini Opticon™ Real-Time PCR Detection System	Bio-Rad	CFX96 Touch™ Deep Well Real-Time PCR Detection System
Roche	LightCycler®480 Real-Time PCR System	Bio-Rad	iCycler iQ™ Real-Time PCR Detection System
Roche	LightCycler®96 Real-Time PCR System	Bio-Rad	iCycler iQ™5 Real-Time PCR Detection System
Agilent Technologies	AriaMx Realtime PCR System	Bio-Rad	MyiQ™ Real-Time PCR Detection System
		Bio-Rad	MyiQ™2 Real-Time PCR Detection System
		Eppendorf	Mastercycler™ep <i>realplex</i>
		Stratagene / Agilent Technologies	Mx3000P™ Real Time PCR System
		Stratagene / Agilent Technologies	Mx3005P™ Real Time PCR System
		Analytik Jena Biometra	TOptical
		Analytik Jena Biometra	qTOWER 2.0

Table A1/A2. Compatible low and high profile Real Time PCR systems.

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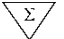
13. Bibliography/Bibliografía

1. C. Schabereiter-Gurtner *et al.* Novel real-time PCR assay for detection of *Helicobacter pylori* infection and simultaneous clarithromycin susceptibility testing of stool and biopsy specimens. *Journal of Clinical Microbiology* 2004; 42(10): 4512-4518.
2. J.G. Kusters *et al.* Pathogenesis of *Helicobacter pylori* infection. *Clinical Microbiology Reviews* 2006; 19(3): 449-490.
3. F. Can *et al.* Urease activity and urea gene sequencing of coccoid forms of *Helicobacter pylori* induced by different factors. *Current Microbiology* 2008; 56(2): 150-155.
4. R.X. Tang *et al.* Diversity of *Helicobacter pylori* isolates in expression of antigens and induction of antibodies. *World Journal of Gastroenterology* 2008; 14(30): 4816-4822.
5. S.K. Patel *et al.* Diagnosis of *Helicobacter pylori*: what should be the gold standard? *World Journal of Gastroenterology* 2014; 20(36): 12847-12859.
6. M. Varbanova *et al.* Chronic gastritis-an update. *Best Practice & Research: Clinical Gastroenterology* 2014; 28(6): 1031-1042.

14. Symbols for IVD components and reagents/Símbolos para reactivos y productos para diagnóstico in vitro

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
VIASURE *Helicobacter pylori* Real Time PCR Detection Kit has been validated on the following equipments: Applied Biosystems 7500 Fast Real-Time PCR System, Applied Biosystems StepOne™ Real-Time PCR System, Bio-Rad CFX96 Touch™ Real-Time PCR Detection System and AriaMx Realtime PCR System (Agilent Technologies). When using the Applied Biosystems 7500 Fast with strips it is recommend to place a plate holder to reduce the risk of crushed tube (Ref. PN 4388506).

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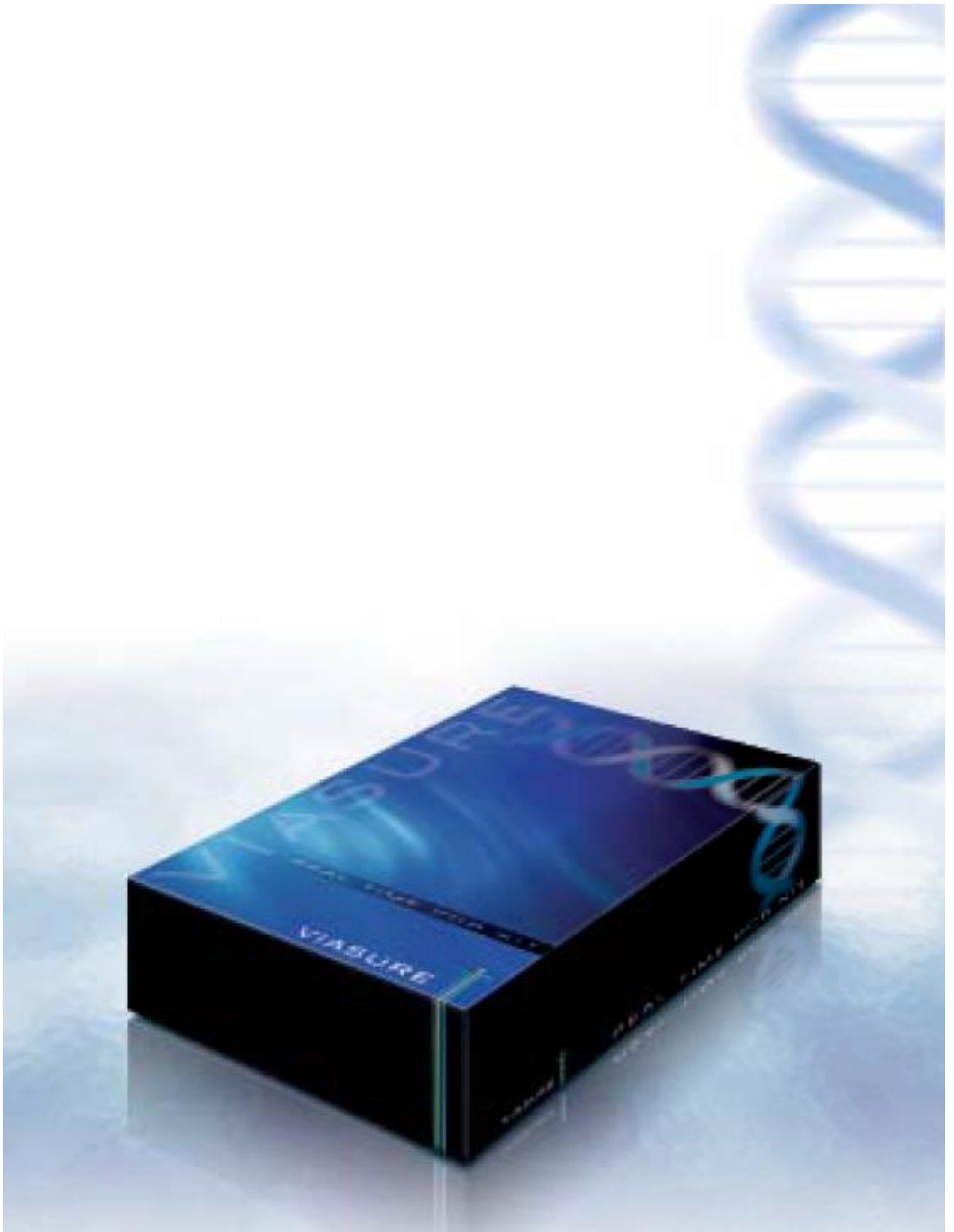- CFX™ and IQ5™ are registered trademarks of Bio-Rad Laboratories.
- ABI®, QuantStudio™, StepOnePlus™ and ViiA™ are registered trademarks of Thermo Fisher Scientific Inc.
- LightCycler® is a registred trademark of Roche.
- Mx3000P™, Mx3005™ and AriaMx Realtime PCR System are registered trademarks of Agilent Technologies.
- Mastercycler™ is a registered trademark of Eppendorf.

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